Vetramil®

THE NATURAL POWER OF HONEY

Instructions for use

Product

Vetramil® honey ointment
10 g / 30 g / 180 g ointment with honey and essential oils for impaired skin

Indication

Vetramil® improves the skin condition and supports the natural healing process of the impaired skin in domestic animals and pets.

Product Performance

Vetramil® is a honey-based ointment with essential oils. The ointment softens and protects dry and irritated skin. Vetramil® contains honey with high enzyme content. In a moist environment the enzyme glucose oxidase affects a slow release of small quantities hydrogen peroxide. Low concentrations of hydrogen peroxide purify the skin and stimulate recovery. The low pH of Vetramil restores normal skin acidity, the essential oils repel insects and prevent animals from licking the ointment.

Instructions for application

1. Unscrew the white cap from the tube.
2. Turn the cap over.
3. Open the tube by pressing the tip of the cap through the tube opening.
4. If necessary, clean the impaired skin with water and dip dry with clean tissue.
5. Apply Vetramil® 1-2 times per day.
6. Massage well onto the irritated skin.
7. Vetramil® can be combined with a gauze dressing.
8. Continue treatment until the skin has fully recovered.

Ingredients:

Paraffinum Liquidum, Mel, Petrolatum, Aqua, Cetearyl Alcohol, Polysorbate 80, Alcohol Denat., Sodium Lauryl Sulfate, Sodium Cetearyl Sulfate, Lavandula Angustifolia Oil, Melaleuca Viridiflora Leaf Oil, Ocimum Basilicum Oil, Pogostemon Cablin Leaf Oil, Thymus Vulgaris Flower / Leaf Oil, Potassium Sorbate, Citric Acid.

Conditions for storage

Storage at Room temperature.

Precautions

- Do not use the ointment if the packaging is damaged.
- Close the packaging after use.
- Avoid contamination of the contents.
- Use the contents within 3 months after opening.
- There is no waiting period after application of Vetramil®.
- Do not use sharp objects to open the tube. This can lead to loosening of pieces of the closing membrane and to inconvenience when using the ointment.

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FI: FaunaPharma Oy, Espoo – www.faanapharma.fi
AT: Vana GmbH, 1020 Wien – www.vana.at
CZ: Dr. E. Graub AG, Bern – www.graub.com