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Bewaren tussen 4°C en 30°C

Instructions for use Revamil Balm Sachet

The Product

Revamil® Balm Sachet
2g ointment for the treatment of superficial wounds

Ingredients

Peanut oil refined, beeswax white, Glyceryl oleate, honey, purified water, rose oil

Product description

Revamil® Balm is a nourishing ointment for the treatment of superficial wounds. Revamil® Balm consists of a base ointment in combination with pure honey. The honey in Revamil® Balm contains high levels of honey enzymes, like glucose oxidase. In a moist environment, this enzyme produces low levels of hydrogen peroxide. The high sugar concentration of honey facilitates mixing of wound moisture with Revamil® Balm, thus forming an intermediate layer between the wound and the honey, which gives anti-bacterial protection. Revamil® Balm does not contain residues from pesticides, herbicides, conservatives or other components that are known to cause allergic reactions.

Instructions for use

1. Open the sachet by tearing it open at the notch
2. Squeeze the ointment out of the sachet and apply a thin layer of Revamil® Balm on the wound
3. The wound can be covered with a wound dressing
4. Treat affected areas once every 24 to 48 hours
5. In case of persisting itching, Revamil® Balm can be applied more frequently

Indications

Revamil® Balm can be applied on superficial wounds. Revamil® Balm is allowed for all age groups

Contra indications

Do not use Revamil® Balm if the patient is known to be oversensitive for one of the ingredients of the ointment. Oversensitivity for one of the ingredients can cause redness, irritation and possibly light pain.

Side effects

- During the first minutes after application, the product can cause a light stinging sensation. The stinging sensation is probably caused by the low pH and high osmolarity of the honey and has no effect on the performance of the product
- In case of unexpected side effects after use contact your doctor or the manufacturer. Side effects can cause inconvenience to the patient.

Safety precautions

- Do not use when the sachet is damaged as this can lead to contamination with bacteria of the product and to wound infection. Because of the antibacterial protection of honey the chance of wound infection is considered very low.
- One sachet is for single use only.
- One sachet is sufficient for only one treatment. Underdosage of the product could lead to lower performance of the product and slower wound healing.
- Do not reuse a sachet. Reuse of a sachet can lead to loss of function of the ointment, contamination of the product and contamination or infection of the wound.
- Do not apply this product for more than 30 days continuously. Treatment for more than 30 days continuously could lead to inconvenience to the patient, like irritation and/or minor pain.
- Do not gamma irradiate a sachet again. Re-gamma-irradiation of an already used sachet could lead to contamination and to wound infection.
- Do not use the product after the expiry date as this can lead to lower performance of the product and to inconvenience for the patient

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Bfactory Health Products B.V.
DK: Axel Madsen A/S, Vedbæk